

MANUAL MODE


To put the wheelchair in manual mode, push down the yellow levers located behind each back wheel. To put back in driving mode pull the handles upwards.



TRANSFERS

Before transferring in or out of the wheelchair, it is important to make sure that these following safety measures are applied.

1. The wheelchair must be **TURNED OFF** before sitting in or standing up from the wheelchair. You must always make sure that all the lights on the control box are **OFF** before transferring in or out of the wheelchair.
2. Before standing, the safety belt must be **unfastened**.
3. The footrests must be swung to the sides in order to free up space in front of the seat. In order to do this, you must press on the **GREY** lever to swing footrests to the sides.

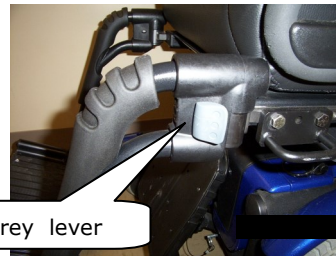

4. You can swing the control box to the side by pushing the lever underneath the control box. Pull back the control box to its original position before operating the wheelchair.



Q 6000

MOTORIZED WHEELCHAIR



Constance-Lethbridge

Rehabilitation Center
7005, de Maisonneuve O.
Montreal Québec
H4B 1T3



TURNING ON

To operate the wheelchair, you must push up the lever located on the left side of the control box.

TURNING OFF

To turn off the wheelchair, you must push down the same lever.



HORN

The button is located just under the screen represented by a yellow horn.

DRIVING WITH THE JOY-STICK

Wheelchair will advance in the direction that you point the joystick. The further you push the joystick, the faster the wheelchair will advance.



SPEED PROFILE INDICATOR

The Q600 has five speed profiles indicated in the bottom right of screen by a number following the letter "P".

CHANGING SPEED PROFILES

To change speed profiles, you must push up the lever on the left side of the control box. The number following the "P" letter will change to show you in which profile you are in.



VARIABLE SPEED KNOB

Speed level is also adjustable within each program by a dial on the right side of the control box and represented by a series of yellow bars in middle of the screen. Your cruising speed will be indicated km/h by a number below yellow bars.

DISPLAY FOR BATTERY CHARGE LEVEL

The display for the battery load remaining in the wheelchair is represented by a series of green squares and also by a percentage number. As load decreases so will squares and percentage number.

BATTERY CHARGE

When half of the green squares of the battery load have lost their color, it is time to recharge the battery.

Connect wire located at the back of the wheelchair and connect it directly in a standard outlet.

To ensure your safety and to avoid damage to the wheelchair, the battery must be charged in one extended period rather than short periods at a time.

